



LAN

Long Alliantie Nederland

Name: _____

Date: _____

Instructor: _____



Inhalation protocol pressurized metered-dose inhaler InspiraChamber® with large mask

Please note: only use the mask if inhalation with mouth piece is not possible

Patient information

- 1. Discuss the purpose and the mode of action of the medication.

Preparation

Before the first use:

- 2. Remove the cap from both the inhaler .
- 3. Shake the inhaler well and spray three doses.

Before every use:

- 4. Remove the cap from both the inhaler.
- 5. If the inhaler hasn't been used for a week, shake the inhaler well and spray one dose.
- 6. The mouth has to be empty.
- 7. Shake the inhaler well.
- 8. Put the inhaler with the opening downwards straight into of the InspiraChamber®.
- 9. The inhaler with the InspiraChamber® is now ready for use.

Inhalation

- 10. Sit or stand up straight with your head slightly backwards.
- 11. Hold the InspiraChamber® horizontal and apply mask to face and ensure an effective seal over mouth and nose.
- 12.
 - Exhale completely.
 - Press down the inhaler canister once at the start of a slow inhalation.
 - **Inhale and exhale slowly 5 times through the InspiraChamber®.** Inhale with less force if the inhaler gives a 'whistle' sound. Make sure the mask has a good seal until the patient is done.
- 13. If an extra dose is prescribed, shake the inhaler with the InspiraChamber® well and repeat steps 7 to 12. Do not press more than one dose into the InspiraChamber®.
- 14. Remove the inhaler from the InspiraChamber®, put the cap back on the inhaler.
- 15. After inhaling, rinse your mouth briefly with water: gargle and spit with the first mouthful then swallow the second mouthful. If needed eat or drink something, and wash the face.

Inhalation protocol pressurized metered-dose inhaler InspiraChamber® with large mask - version 2.0



LAN

Long Alliantie Nederland



Cleaning

- 16. Clean the InspiraChamber® once a week.
- 17. Dismantle the InspiraChamber®. Remove the backpiece and the mask of the InspiraChamber®.
- 18. Wash all parts of the inhaler in lukewarm, (clean) soapy water.
- 19. Rinse the InspiraChamber® but do not dry it.
- 20. Leave the InspiraChamber® to air dry.
- 21. Make sure the components are dry before the InspiraChamber® is assembled.

Important points

- 22. Estimate when the inhaler will be empty. The number of doses which the inhaler contains is listed on the side of the inhaler. A suggestion is to tally the doses and put down the date of embarking. Some inhalers have a dose counter to tell you exactly how many doses are left.
- 23. Keep the inhaler at room temperature.
- 24. Replace the InspiraChamber® once a year.

Finally

- 25. Have a look at an instructional film which shows these steps in action.

Inhalation protocol pressurized metered-dose inhaler InspiraChamber® with large mask - version 2.0

These English protocols are based on the inhalation protocols of the Dutch Lung Alliance and are being translated by the ADMIT group www.admit-inhalers.org.

The Dutch Lung Alliance encourages dissemination of the protocols and the instructions. The most recent versions and terms of use can be found at the website www.inhalatorgebruik.nl